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Pulittitie 14
00880 Helsinki
info@high-adventure.fi
p.0405630093
p.0407051672

Critical safety information

Always ensure the safety of the product before use. Make sure the safety belt lock is working properly before use. When tightening the safety belt, it must be taken into account that the breathing movements of the patient's chest are not obstructed. The safety belt must not be placed over the patient's neck. The safety belt should be tightened so that there are no threads, or the edge of the belt can press on the patient's body. Always monitor the safety of the product during use. The HARRT safety belt is intended to be attached only to approved body, rescue stretcher and carrier models. The product is intended for use by a person or group with sufficient training. The safety belts should always be taut so that the weight of the person lying on the stretcher is distributed evenly on them, if the position of the basket or rescue stretcher is changed, it is turned or it turns over unexpectedly. The safety belt may only be attached to sufficiently durable attachment points indicated in the user manuals by the manufacturer of the basket and rescue stretcher. Protects product straps from sharp and cutting edges and surfaces. Avoid contact of chemicals with the product. Do not handle the product near an open flame.

Product markings

- *Manufacturer
- *Date of manufacture
- *CE
- *Class-I medical device EU2017/745
- *Breaking strength: MBS 1300 kg

Durability and breaking strength of the product

Safety belt breaking strength: MBS 1300 kg. The breaking strength of the connection point must be verified before fastening safety belt. Use only the attachment points indicated or approved by the manufacturer of the basket or rescue stretcher for attaching safety belts. The product can be used in the temperature range of -30 to +50°C.

Product overview (PICTURE 1.)

Openable and quick-locking safety belt to attach to body and rescue stretcher. The purpose of the safety belt is to ensure the patient's safe fastening and staying in place during carrying on a basket or rescue stretcher and during rescue operations.

- A-Safety belt fastening loop
- B-Safety belt brand mark
- C-Safety belt opening and adjustable quick release with push button
- D-Belt adjustment allowance
- E-Plastic securing loop for the free end of the Belt

Attaching the product to a basket or rescue stretcher (PICTURE 2.)

Product maintenance and cleaning

The product should be kept clean and cleaned regularly. The product can be washed with mild soapy water and should be rinsed clean with plenty of water. Do not use bleach. Maximum washing water temperature +30°C. Do not wash the product in the washing machine. Dry the product carefully before use. Do not machine dry the product. The product should be stored in a cool place away from sunlight and air pollutants such as exhaust gases, vapors and dust. The product may not be repaired or changed without the permission of the manufacturer.

Checking the product

The product must be inspected before and after use to ensure its safety. The product must be inspected by a trained inspector every 12 months at the latest. An inspection diary must be kept for inspecting the product.

Before each use, check the product's straps for cuts, wear affecting use, traces of melting or chemical damage. Make sure the adjustment buckles are functional and that there are no bends or breaks in the metal parts of the buckles. Check the load bearing stitches of the belts for tears or damage to the stitches. A damaged product must not be used.

Warranty

The product has a 12-month warranty against manufacturing defects. The warranty does not apply to material damage caused to the product by the user during use or damage caused by misuse of the product.

Product storage and lifespan

Storage time 2 years + 10 years life in service. The condition of the product always deteriorates during use, due to the abnormal nature of the operating environment. Therefore, it is impossible to determine the exact service life.