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p.0407051672

Critical safety information

The HARRT Pro RS auxiliary strap provides additional security for attaching a person to the rescue stretcher during rescue and training operations. The RS strap must not be used as the sole securing device. The RS strap is not fall protection or part of a fall prevention system, but an aid providing additional safety. Always ensure the safety and working condition of the product before use. Constantly monitor the safety of the product during operation. The HARRT Pro RS Up and Down adjustable strap is intended to be connected to body and stretcher models approved for rope rescue operations. Using the strap safely requires special skills. The product is intended only for use by a person or group who is familiar with the use and sufficiently specially trained. The product must not be used other than as described in this instruction manual. The V-shaped opening angle formed by the strap should be kept to a minimum to avoid overloading. The strap should be adjusted so that the weight to be supported is evenly distributed over all parts of the strap. The strap may only be attached to sufficiently robust anchorages of the basket and rescue stretcher. Protect the strap from sharp and slashing edges and surfaces. Avoid getting chemicals into the product. Do not handle the product near an open flame. The product can be used in the temperature range from -30 to +50°C.

Product markings

*Product Title

*Manufacturer

*Date of manufacture

Product overview

HARRT Pro RS – auxiliary strap (FIG. 1).

A- Stitched mounting loop

B- Belt connection lock and adjustment buckle

C-brand

HARRT Pro RS – Installing the straps on the rescue stretcher (FIG. 3).

RS straps are the same length, and in order to keep the patient up, for example, a change of stretcher posture or in situations where the stretcher rotates, it is recommended to use two RS straps on the stretcher. One is installed on the main side of the body stretcher and the other on the foot.

HARRT Pro RS – attaching auxiliary straps to the patient (FIG. 4).

RS straps should be attached to approved fall protection or restraint attachment clips on the patient evacuation or work harness. The straps can be fixed at the same point. RS straps should be fastened on the head side so that, if possible, the straps run on both sides of the patient's head. At the foot end, the straps should be attached to the outside of the feet on both sides so that they support the feet as effectively as possible. To attach the auxiliary strap to the patient's harness, closed rings approved for the operation can be used.

Maintenance and cleaning

The product should be kept clean and cleaned regularly.

The product can be washed with mild soapy water and should be rinsed clean with plenty of water. Maximum washing water temperature +30°C. Do not wash the product in the washing machine. Do not machine dry the product.

The product should be stored in a cool area, protected from sunlight and air contaminants such as exhaust gases, vapours and dust.

The product may not be repaired or modified without the permission of the manufacturer.

Product inspection

The product should be inspected before and after use to ensure safety.

Check the product straps for cuts, wear affecting use, melt marks, or chemical damage. Make sure that the adjustment buckles work properly and that there are no twists or cracks on the metal parts of the buckles. Check the load-bearing stitches of the straps for tears or damage to the stitches.

Damaged product must not be used.

Guarantee

The product has a 12-month warranty against manufacturing defects. The warranty does not apply to material damage caused by the user to the product during use or damage caused by misuse of the product.

Product life

Storage time 2v + 10v service life

The condition of the product always deteriorates when used due to the abnormal nature of the operating environment. Because of this, it is impossible to determine the exact service life.