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Suitable for use with one- and two-piece basket and rescue stretchers.

High Adventure Oy

Pullittie 14
00880 Helsinki
info@high-adventure.fi
p.0405630093
p.0407051672

Critical safety information

The HARRT Pro Restraint System patient harness provides additional security for securing a person to a basket stretcher during rescue and training operations. The Restraint System should not be used as the only means of securing a patient. The Restraint System is not a fall arrester or part of a fall arrest system, but an auxiliary device providing additional security.

Always ensure the safety and functional condition of the product before use. Do not use a damaged product.

Monitor the safety of the product continuously during use.

The HARRT Pro Restraint System is intended to be connected to basket and rescue stretcher models approved for rope rescue operations.

Safe use of the harness requires special skills.

The product is intended only for use by a person or group who is familiar with its use and has received sufficient special training.

The product must not be used in any way other than as described in this manual. The attachment straps should be adjusted so that the weight of the person lying on the stretcher is evenly distributed across all parts of the strap. When using the product, it should always be taken into account that the person lying on the stretcher is able to breathe normally after tightening the strap. The fastening strap may only be attached to sufficiently durable attachment points on the basket and rescue stretcher.

Protect the strap from sharp and cutting edges and surfaces.

Always secure the free parts of the strap so that there is no risk of getting caught or entangled. Avoid getting chemicals into the product.

Do not handle the product near open flames.

The product can be used in the temperature range of -30 - +50°C.

Maximum patient weight when using the product WLL 280kg 8:1

Product markings

- *Product name
- *Manufacturer
- *Manufacturing date
- *Approval information

Product overview

HARRT Pro Restraint System (FIG. 1).

- A- Long green lashing strap 1 pc
- B- Long red lashing strap 1 pc
- C-Long strap locking buckles and lashing loop 2 pcs
- D-Lashing loop with O-ring 8 pcs (4 pcs green and 4 pcs red)
- E-Long strap lashing loop

HARRT Pro Restraint System – attaching to the stretcher (FIG. 2).

The HARRT Pro -Restraint System patient harness must be attached to the load-bearing structural parts of the basket pair. The long straps (red and green) are attached parallel to the ends of the stretcher (Figure 2.) so that the locking buckles of the straps are on the head side of the stretcher. The long straps are attached to the foot and head sides of the stretcher as shown in Figure 2.1. When installing the harness, it must be noted that the installation must be done in such a way that the patient is not injured when the straps are tightened during use. The O-ring strap loops must be attached to the load-bearing structures of the stretcher. The O-ring strap loops are installed on the stretcher structures by attaching them as shown in Figure 2.1. Depending on the stretcher model, the straps are attached at equal intervals to the side structures of the stretcher on the inside of the stretcher. The pre-assembled harness should follow a "Zig-Zag" pattern, as shown in Figure 2. The straps should be attached so that the different colored (green and red) straps are attached at the same point on both sides of the stretcher, so that the patient's weight is evenly distributed on the harness during use.

HARRT Pro Restraint System – releasing, attaching and tightening the strap during use (FIGURE 3).

Before placing the patient on the rescue stretcher, the harness must be released. This is done by opening the locking buckles of the long straps on the head side and pulling the long straps through the O-ring fastening straps. Both long straps must be released from all O-ring fastening straps. The O-ring fastening straps are not removed from the stretcher structures. The patient is placed on the stretcher in a lying position so that no parts of the harness are under the patient. The harness is secured over the patient by passing the long strap through the O-rings of the fastening straps of the same color on the sides of the stretcher. The green long strap passes through the O-rings of the green fastening straps. The red long strap passes through the O-rings of the red fastening straps (Figure 2). The long straps are locked into the locking buckles of the same color strap parts on the head side of the stretcher. Ensure that the pattern of long straps of different colours formed on the patient is symmetrical. Tighten the straps by pulling the strap passing through the locking buckle on the head side. Ensure that the straps of different colours are equally tight. The final tightening should be done so that the patient can breathe freely. This is ensured by tightening during the patient's deep inhalation phase. Ask the patient to breathe deeply or ensure that the chest expands mechanically to ensure sufficient patient safety. The harness should be positioned so that the long straps on the head side are attached to different sides of the patient's head without causing a risk of crushing. When using the harness, it is recommended to wear a safety helmet to protect the patient's head. The foot straps should be positioned so that they support the patient's lower limbs and allow the feet to be in an upright position.

Maintenance and cleaning

The product should be kept clean and cleaned regularly.

The product can be washed with mild soapy water and rinsed thoroughly with plenty of water. Maximum washing water temperature +30°C. Do not wash the product in a washing machine. Do not dry the product mechanically.

The product should be stored in a cool place, protected from sunlight and air pollutants such as exhaust gases, fumes and dust.

The product must not be repaired or modified without the manufacturer's permission.

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Guarantee

The product has a 12-month warranty against manufacturing defects. The warranty does not apply to material damage caused by the user to the product during use or damage resulting from misuse of the product.

Product life

Storage period 2 years + 10 years service life

The condition of the product deteriorates with each use due to the varying nature of the operating environment. Therefore, it is impossible to determine the exact service life.